



Techno-Aide, Inc.
7117 Centennial Boulevard
Nashville, TN 37209
P: 800.251.2629
F: 800.554.6275

X-Ray Markers & Rulers CLEANING & CARE INSTRUCTIONS 2018

Thank you for choosing Techno-Aide for your radiology accessory needs. To ensure the longevity of your new X-Ray Markers / Ruler, we recommend the following guidelines for cleaning and care.

Cleaning and Care

Do:

- Techno-Aide recommends a thorough cleaning with an alcohol free disinfectant wipe such as PDI Sani-Cloth Germicidal Wipes Cat# [GWP-WAF](#) before initial use.
- To disinfect the surface of your x-ray markers or x-ray ruler after patients, Techno-Aide recommends the use of an alcohol free disinfectant wipe such as PDI Sani-Cloth Germicidal Wipes Cat# [GWP-WAF](#). Allow a drying time of at least 3 minutes.

Don't:

- Do not use pure Bleach or any harsh chemicals on the surface as it may weaken the epoxy and cause the polystyrene casing to become brittle over time.

Failure to adhere to the above recommendations can shorten the life of your x-ray marker / ruler. Techno-Aide provides a one year warranty on workmanship and materials.