

CT Slickers & Slicker Cushions CLEANING & CARE INSTRUCTIONS 2018

Thank you for choosing Techno-Aide for your CT accessory needs. To ensure the longevity of your new slicker/cushion, we recommend the following guidelines for cleaning and care.

Cleaning & Care:

We all know that in a medical environment, it is easy for vinyl's to become dirty and soiled from blood, bodily fluids, betadine, and other fluids.

Do:

- Slicker covers and cushions are not sterile when shipped. We suggest a thorough cleaning with a bleach/water solution as recommended by the center for disease control before each use.
- Techno-Aide recommends cleaning with cold water and a mild detergent immediately after contact with fluids. Household 409 Spray can be used to clean the vinyl and remove dust and dirt from your slicker or slicker cushion
- To disinfect your table pad, Techno-Aide recommends the use of disinfectant wipes such as PDI Sani-Cloth AF3 Alcohol Free Germicidal Wipes Cat# GWP-WAF. Allow a drying time of at least 3 minutes Store your table pad on a pad rack when not in use.
- Some Germicide solutions may cause discoloration of the cover and create a sticky film.
- Adhesive remover or nail polish remover will clean off tape residue easily without harming the cover.

Don't:

- Do not autoclave or launder.
- Do not soak/submerge slicker/slicker cushion in water.

Failure to adhere to the above recommendations will shorten the life of your slicker / slicker cushion. Techno-Aide provides a 90 day warranty on workmanship and materials for our slickers / slicker cushions. In normal use, a replacement cycle of one year is recommended.