



Techno-Aide, Inc.  
7117 Centennial Boulevard  
Nashville, TN 37209  
P: 800.251.2629  
F: 800.554.6275

---

## Table Drape CLEANING & CARE INSTRUCTIONS 2018

Thank you for choosing Techno-Aide for your radiation protection needs. To ensure the longevity of your new table drapes, we recommend the following guidelines for cleaning and care.

### **Cleaning:**

We all know that in a medical environment, protective apparel nylon fabrics can become dirty and soiled from blood, bodily fluids, betadine, barium, and other fluids.

### **Do:**

- Techno-Aide recommends cleaning with cold water and a mild detergent to remove all immediate contacts with fluids.
- 409 Spray can be used to clean the surface fabric and remove dust and dirt from your garment.
- To disinfect your lead shield, Techno-Aide recommends the use of disinfectant wipes such as PDI Sani-Cloth Germicidal Wipes Cat# GWP-WTAF. Allow a drying time of at least 3 minutes.

### **Don't:**

- Do not use Bleach or any harsh chemicals on your garment.
- Do not autoclave
- Do not soak/submerge garment in water.
- Do not launder.

Failure to adhere to the above recommendations will shorten the life of your shield, promote cracking, and possibly void your warranty. Techno-Aide provides a two year warranty on workmanship and materials for our protective lead shields



Techno-Aide, Inc.  
7117 Centennial Boulevard  
Nashville, TN 37209  
P: 800.251.2629  
F: 800.554.6275

---

**CARE:**

Radiation protective apparel and shields are sometimes inadvertently damaged when not properly handled or stored. The following guidelines may be used to help achieve this goal:

**Do:**

- Before use, inspect for defects, tears, perforations, creases.

**Don't:**

- Don't exposure your shield to extreme temperatures of hot or cold or to direct sunlight.
- Properly fold not wad drapes when not in use and lay on flat surface
- Do not lean drapes against sharp edges or objects with sharp points.

Failure to adhere to the above recommendations will shorten the life of your shield, promote cracking, and will void your warranty. Techno-Aide provides a two year warranty on workmanship and materials for our protective apparel garments

**DISPOSAL:**

Under the Resource Conservation and Recovery Act (RCRA) and the Environmental Protection Agency (EPA) Lead is included in the Toxic substance list. It should not be disposed of as conventional garbage but as hazardous waste. Coordinate with the radiation control officer in your organization who is responsible for disposal of such wastes. Techno-Aide recycles only Techno-Aide manufactured apparel and shields. If you currently have a TA product that needs to be disposed of, simply ship it back to us and we will dispose of it properly at no cost to you (other than shipping). LMG-Lead Free Drapes do not need to be recycled as they are not toxic to the environment and can be thrown in regular trash after they exceed their life cycle.

**If you have any Questions regarding your Techno-Aide Radiation Protection Shield Purchase, please contact your Local X-Ray Dealer or Techno-Aide Customer Service.**